

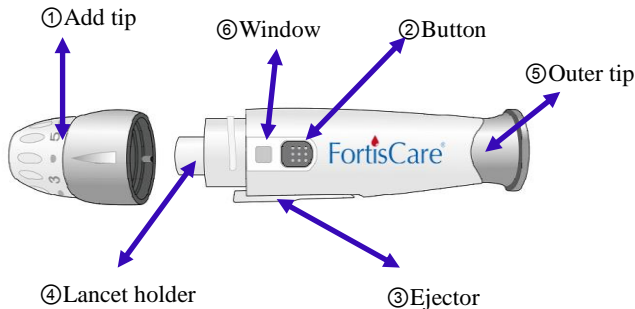
INSTRUCTIONS FOR USE

The **FortisCare Pain Management Lancing Device** is a convenient medical tool for collecting capillary blood samples for glucose monitoring or other tests which require drops of blood. As a more advanced lancing device, it provides ultimate safety and comfort for obtaining blood samples. An adjustable tip offers 7 levels of skin penetration for individual user's comfort and a lancet ejector enables the safe disposal of the used lancet.

PRECAUTIONS

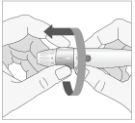
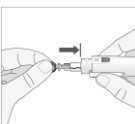
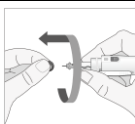
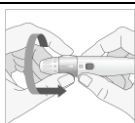
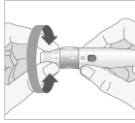
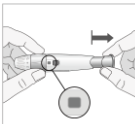
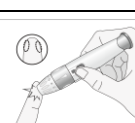
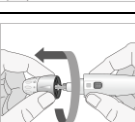

- Never use a lancet that has been used by someone else. This could lead to contamination.
- If the lancing device is to be used by another person, the unit must be properly disinfected and a new tip and lancet must be used.
- Do not leave the lancet in the device after use.
- A new lancet must only be placed into the lancing device directly before testing.
- This device has many small parts and could be a choking hazard for children if swallowed. Keep away from infants and children!
- In the case of hospital use, hospitals need to consult their own infection control protocols in order to avoid any contamination.
- Users clean the tip after using it. This can avoid contamination.
- Always dispose of the used lancet in a biohazard container.

COMPONENTS



HOW TO USE FortisCare Lancing Device

Wash your hands carefully with warm water to increase the circulation of the blood into the fingers. Dry your hands thoroughly until the finger to be pricked is completely dry. Please read the following steps with corresponding illustrations to understand how to use the **FortisCare Lancing Device**.

	<p>① Unscrew the add tip of the lancing device by turning it counter-clockwise with holding the device firmly.</p>
	<p>② Insert a new sterile lancet to its end into the lancet holder.</p>
	<p>③ Twist off the protective cap of the lancet.</p>
	<p>④ Screw the add tip of the lancing device by turning it clockwise.</p>
	<p>⑤ Set the desired penetration depth from 1 to 7 by turning the add tip. 1-3 for soft or thin skin, 4 for average skin, 5-7 for thick or calloused skin.</p>
	<p>⑥ Hold the device firmly with one hand then load the lancing device by pulling out the outer tip with the other hand. The window turns to red with loading.</p>
	<p>⑦ Place the lancing device on the site of the finger tip. Hold the lancing device against the finger tip without too much pressure. (The harder it is pressed the deeper the puncture.) Press the release button and take off the lancing device to take blood sample.</p>
	<p>⑧ Unscrew the add tip of the Lancing device by turning it counter-clockwise.</p>
	<p>⑨ Push the lancet ejector forward with the thumb until lancet is removed. Dispose of the used lancet into a proper biohazard container.</p>

RECOMMENDED AVAILABLE LANCETS

Some general sterilized lancets are BD-Ultra Fine, Cleanlet Fine, E-Z Ject, G-P Lite, Microlet, Monolet, One Touch, Soft Touch, Therasense, Ultra TLC and Unilet GP.

Note: Some less known lancets may not work properly with the **FortisCare lancing device** due to their dimensions. Please, contact your supplier to make sure you are using proper lancets.

STORAGE

Products must be stored at room temperature protected from sunlight or moisture.

CLEANING AND DISINFECTION

Clean and wipe the outside of the **FortisCare lancing device** a minimum of once a week with a soft cloth, dampened with mild soap and water. Wash the adjustable tip and the clear tip once a week with mild soap. DO NOT immerse the **FortisCare lancing device** in liquid.

WARRANTY

FortisCare lancing device has a 2-year warranty from the date of purchase. A malfunctioning device should be returned to the dealer you purchased your lancing device from or contact us:

Contact:

Royal Medical Supplies

www.royalmedicalsupplies.com

E mail: contact@royalmedicalsupplies.com

